

Reinigung des Medico Panels

Hier finden Sie eine Liste mit zugelassenen Mitteln um unsere Medico Panels zu reinigen:

Cleaning	WMP-17X Touch	WMP-17X Housing
40% Sodium Hydroxid	Moderate White Discoloration- Not support	Not support
Acetone	OK	Not support
Carbitol Acetate	OK	Not support
Cellosolve Acetate	OK	Not support
Ethanol -Alcohol	OK	500g load with 95% Alcohol--> OK
Methylene Chloride	OK	Not support
Hydrochloric Acid	OK	Not support
Hexane- 正己烷	OK	Not support
Isopropanol (Isopropyl Alcohol)/Propanol	OK	Not support
Methyl Ethyl Ketone	OK	Not support
Heptane	OK	Not support
Toluene	Not support	500g load with 5% Cresol --> OK
Turpentine	OK	Not support
Vm & P Naptha	OK	Not support
Bleach (Sodium Hypochlorite , NaOCl)	Moderate White Discoloration- Not support	500g load with 10% Bleach--> OK
Benzyl alcohol	OK	Not support
Hydrogen peroxide	12% Hydrogen peroxide > 20hrs --> Damage 3% Hydrogen peroxide with short time contact -->OK	Not support

Benötigen Sie weitere Hilfe?



Kontaktieren Sie für Pre-Sales Anfragen bitte unseren Vertrieb.

Falls Sie Unterstützung zu einem erworbenen Produkt benötigen, kontaktieren Sie bitte unsere Support Abteilung.

Bitte halten Sie Ihre Kundennummer und Rechnungsnummer bereit, um unnötige Wartezeiten zu vermeiden.

Unsere Kontaktdaten:

- [Vertrieb](#)
- [Support](#)